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
# CARNET D'ENTRETIEN SERVICING & MAINTENANCE MANUAL

## MINI MINOTOR 3, MINOTOR 3, MINOTOR 3 XL LIGHT DRIVE MINI 3, LIGHT DRIVE 3, LIGHT DRIVE 3 XL

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# CONTENTS

<b>GENERAL REMARKS .....</b>	<b>24</b>
Intended use .....	24
Indications and ability .....	24
Service life .....	25
Regulations .....	25
Symbols used in this manual .....	25
Meaning of symbols appearing on the labels .....	25
<b>SPECIFICATIONS OF LIGHT DRIVE MINI 3/ LIGHT DRIVE 3/ LIGHT DRIVE 3 XL ...</b>	<b>26</b>
General assembly view of the power units .....	26
Technical specifications .....	26
Components .....	27
<b>DRIVING PRACTICES .....</b>	<b>29</b>
How to drive ? .....	29
How to approach a slope or an obstacle ? .....	29
How to tackle ground sloping ? .....	30
How to brake ? .....	30
How to switch to manual mode ? .....	30
<b>SERVICING AND MAINTENANCE .....</b>	<b>31</b>
Servicing the equipment .....	31
Performance checks .....	31
Waste management .....	32
Troubleshooting .....	33
Control unit fault code indicator .....	34

<b>TRANSPORT AND STORAGE .....</b>	<b>35</b>
Transport .....	35
Storage .....	35
<b>SAFETY AND HAZARD INSTRUCTIONS .....</b>	<b>36</b>
General safety .....	36
Safety information about the electrical system .....	37
Safety recommendations about electromagnetic compatibility .....	38
Safety information about installation .....	38
Safety information about driving .....	38
Safety information about servicing and maintenance .....	39
Safety information about modifications made to the equipment .....	39
Important information about the maintenance tools .....	40
CE marking .....	40
Important legal notice for the user of this product .....	40
<b>GUARANTEE .....</b>	<b>41</b>
Conditions .....	41
Limits of liability .....	41

# GENERAL REMARKS

Read this servicing & maintenance manual before using this product and keep it for future reference.

The content of this document may change without notice. We recommend you visit our website regularly for any changes or updates.

## INTENDED USE

Combining precision, ease-of-use and safety, Light Drive range is a lightweight power add-on that transforms a manual wheelchair into a power wheelchair, making it easier to move about in everyday life. It is designed for anyone whose ability to walk has been impaired, but who, having good eyesight and being physically and mentally sound, is able to drive a motorised vehicle. Designed to meet the practical needs of users seeking independence and mobility, it will be the ideal and reliable companion to take you everywhere, both indoors and outdoors, even on difficult ground. Nonetheless, it is not intended to convert the wheelchair into an all-terrain or sports wheelchair.

Users must follow the general safety and use operating instructions. As the manufacturer, BENOIT SYSTEMES reserves the right to change product specifications without notice.

**i** Please avoid using the LIGHT DRIVE range before receiving instructions about the equipment from an approved dealer.

For any question about the use, servicing or safety of your device, contact your dealer.

If you have further questions, you can contact:

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[export@benoitsystemes.com](mailto:export@benoitsystemes.com)

## INDICATIONS AND ABILITY

The LIGHT DRIVE 3 range is compatible with most manual wheelchairs (see the compatibility guide on our website) and suits people unable to walk or with restricted mobility due to:

- Paralysis,
- Loss of a limb (amputation of the leg),
- A limb problem or deformity,
- Spasms / injuries to the joints,
- Strokes and brain injuries,
- Neurological disorders (multiple sclerosis, Parkinson's, etc.),
- Illness, such as heart and circulatory disease, balance disorder, as well as elderly people with enough strength in the upper body,
- People mentally and physically capable of using a control unit to handle the motorised wheelchair and its features safely

The prescription conditions must consider all factors for which the LIGHT DRIVE 3 range provides a relevant answer. A PM&R physician (Physical and Rehabilitation Medicine), assisted by an occupational therapist and a physiotherapist, will assess the suitability of the adaptive comfort solution with regard to the patient's condition and abilities, environment, and physical and cognitive capacities.

A validation test will be carried out in the presence of the multidisciplinary team and the patient. The purpose of this test will be to confirm the best match between:

- The physical and cognitive capacities of the patient, which should enable him to safely handle the device,
- And the device itself to meet the need of this particular disability compensation.

## SERVICE LIFE

BENOIT SYSTEMES considers a service life of its products at 5 years, as long as the product is used for its intended purpose and obeying use, safety, and maintenance instructions. However, this service life can be exceeded when the product is serviced and maintained carefully and used cautiously, appropriate to its technical specifications.

The service life may also be considerably shortened following extreme or inappropriate use. The service life set by BENOIT SYSTEMES is for information only and does not constitute an equivalent or additional contractual guarantee.

## REGULATION

The LIGHT DRIVE range, designed by BENOIT SYSTEMES, is considered a "Medical Device" in accordance with Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulations (EC) No 178/2002 and No 1223/2009, and repealing Council Directives 90/385/EEC and 93/42/EEC.

## SYMBOLS USED IN THIS MANUAL

In this manual, hazard statements are indicated by symbols. These symbols are accompanied by a warning indicating the risk level.

### WARNING

Indicates a dangerous situation that, if not avoided, is likely to cause severe or even fatal injuries.

### CAUTION

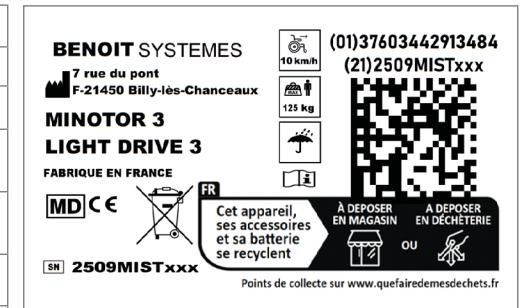
Indicates a hazardous situation that, if not avoided, is likely to cause in minor or slight injuries

### INFORMATION

Gives advice, recommendations and useful information for efficient, problem-free use.

## MEANING OF SYMBOLS APPEARING ON THE LABELS

	Manufacturer
	Medical device
	CE marking in compliance with Annex V of European Regulation 2017/745/EEC
	Power add-on serial number. It is located under the power add-on, between the battery brackets and on the first page of the servicing & maintenance manual.
	Read the instructions for use carefully - indicates the need for the user instructions and safety guidance carefully.
	Keep dry
	Maximum speed
	Maximum user weight
	This product contains parts that can be recycled and substances potentially harmful to the environment. It must be directed to appropriate waste treatment streams (see "Waste management," page 32).
	UDI (Unique Device Identification) number enabling traceability of medical devices.


# SPECIFICATIONS OF LIGHT DRIVE MINI 3/ LIGHT DRIVE 3/ LIGHT DRIVE 3 XL

## WARNING

It is prohibited to modify the equipments without the producer's authorisation.
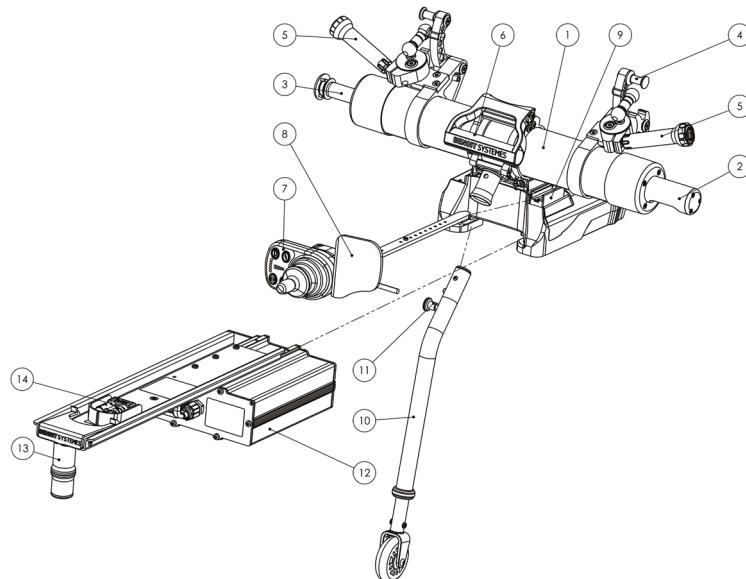
## GENERAL ASSEMBLY VIEW OF THE POWER UNITS

## TECHNICAL SPECIFICATIONS

	LIGHT DRIVE MINI 3	LIGHT DRIVE 3	LIGHT DRIVE 3 XL
Weight of the power unit (kg)	7,5	7,5	9,0
Min. dimensions (mm)	Width: adjusts to the width of the wheelchair without extending beyond the handrims Depth: 280 Height: 270		
Max. user weight* (kg)	80	125	150
Max. speed (km/h)	10	10	6
Max. safe slope (°)	6	6	4
Max. crossable obstacle height (mm)	50	50	50
Suitable wheelchairs**	Compatible with most manual wheelchairs		

\* If you exceed this limit, you might damage the frame or fastenings of your equipment, which could cause serious injuries for you or someone else. Ensure the combined weight of the user and the device does not exceed the allowable load of the manual wheelchair (refer to the manual wheelchair's technical specifications).

\*\* A compatibility guide is available on our website to know the wheelchair models on which the Light Drive 3 can be installed, in compliance with the general safety and performance requirements set out in European Regulation 2017/745 on medical devices.

## COMPONENTS

The power add-on consists of several components :

- |                                 |  |
|---------------------------------|--|
| ① Power unit                    | ⑧ Handrest                             |
| ② Roller covers                 | ⑨ Storage bracket for the control unit |
| ③ Drive rollers                 | ⑩ Anti-tip                             |
| ④ Axles                         | ⑪ Pin locker                           |
| ⑤ Handle                        | ⑫ Battery                              |
| ⑥ Carrying and unlocking handle | ⑬ Battery handle                       |
| ⑦ Control unit                  | ⑭ Power connector (red)                |

## POWER UNIT (1)

It consists of two electric motors equipped with drive rollers ③, clutch mechanisms, and the battery support.

## POWER SUPPLY

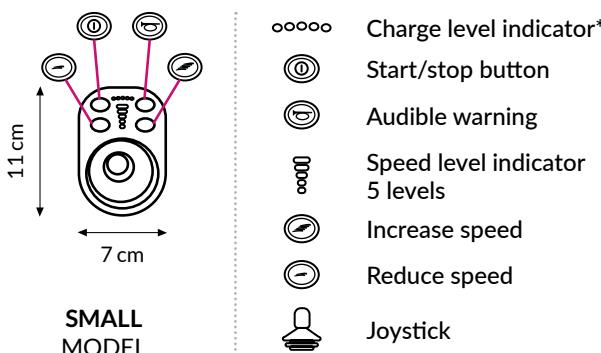
The power supply consists of a 282 or 484 Wh lithium-ion battery ⑫, installed on a bracket that enables automatic installation and connection to the power unit.

It is delivered charged and turned off, except when transported by air.

We decline all liability for damage caused by using other batteries not supplied by us.

A battery user manual is supplied with the product to assist with commissioning and maintenance.

## CONTROL UNIT (7)

\* When the first green LED go out on the charge level indicator, 40% capacity remains. The number of illuminated LEDs is proportional to the remaining capacity.

## ANTI-TIP (10)

It must always be set up, wheel bottom facing. It has been adjusted by your technician to ensure safe driving while allowing obstacles to be crossed up to 5 cm.

⚠ BENOIT SYSTEMES waives all liability if an accident occurs while the anti-tip is not fitted.

## ACCESSORIES (OPTIONAL)

BENOIT SYSTEMES offers various optional accessories such as:

- **Mudguards**, an essential accessory for outdoor trips in bad weather.
- **An attendant control unit**, a complementary control unit allowing an attendant to control the power add-on instead of the user.
- **Various joystick tips for the control unit** to suit user needs and abilities.
- **A car charger with cigarette-lighter plug**, convenient for charging the battery when traveling.

Only original BENOIT SYSTEMES accessories must be used.

For more information on accessories, please contact your dealer.

## DRIVING PRACTICES

Before making your first journey, you should familiarise yourself with operation of the motorised vehicle and all the accessories. Take your time to test all the functions and driving modes.

### PROPERLY SEATED = SAFE DRIVING

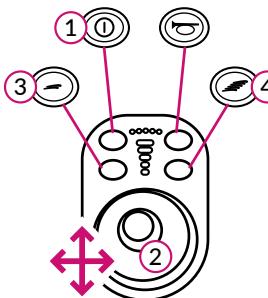
Before each journey, check that:

- You are seated so as to have access to all the controls.
- The battery is sufficiently charged for the journey you're planning to make
- The safety anti-tip is clipped into the rear handle. Check the mounting direction (small wheel downwards) and that the locking pin is securely fitted.
- Your tires are correctly inflated (recommended pressure 8 bar).

### ⚠ WARNING

Do not use any other motorised device or accessory at the same time as the power add-on. If you wish to use another device, please remove the power add-on from the wheelchair for your own safety.

### HOW TO DRIVE ?

Switch on by pressing the control unit **On/Off button** [1] once. The device is then powered on. Pressing the button a second time turns the control unit off. Any malfunction will be indicated by flashing lights on the control unit, which prevents the wheelchair from moving.

The direction and speed are controlled by the **joystick** [2]. Although the control system is relatively easy to use, a little practice will be needed to control the wheelchair properly at different speeds.

To move, just push the joystick in the direction you want to go. The joystick's distance from its neutral position determines the speed and radius of curvature of movement

When the joystick is released it returns to the neutral position (central) automatically, the wheelchair's movement stops and the brakes are applied.

The speed the wheelchair can reach is set by the speed regulator. The user can select up to 5 possible speeds levels using buttons [3] or [4].

Each level represents about 20% of the device's maximum speed.

### HOW TO APPROACH A SLOPE OR AN OBSTACLE ?

The power add-on is designed for safe driving within the slope and obstacle limits stated in the technical specifications.

Approach climbs with sufficient speed and a fully charged battery. Effort can cause a voltage drop leading to progressive stop due to an insufficient charge level.

On a hill start, push the joystick progressively to avoid abrupt movement of the wheelchair, which can cause rearward imbalance. If necessary, lean forward to prevent the front casters from lifting off the ground.

Reduce speed downhill. If the battery is fully charged and speed is excessive, an overvoltage may occur and cause a sudden stop.

Approach any obstacle perpendicularly, preferably in reverse, so both wheels climb simultaneously. First ensure its height is compatible with the anti-tip or the wheelchair's capability.

Never remove the anti-tip to climb an obstacle alone.

#### ⚠ WARNING

BENOIT SYSTEMES accepts no liability for accidents resulting from incorrect installation or positioning of the anti-tip.

## HOW TO TACKLE GROUND SLOPING SIDEWAYS ?

When the ground slopes sideways, the wheelchair is drawn naturally towards the lower side. For this reason, you should correct the direction using the joystick, particularly anticipating it as much as possible on steep climbs. In some difficult routes, another person's help may be useful.

Reduce speed downhill.

Never traverse steep slopes sideways to avoid the risk of the wheelchair stopping or lateral tipping.

## HOW TO BRAKE ?

To prevent any uncontrolled movement of the motorised vehicle, the motors are fitted with automatic brakes that are applied under the following conditions:

- The start/stop switch is in the 'stop' position.
- The start/stop switch is in the 'start' position and the joystick is in the neutral position.
- Power supply is interrupted.

The brake is released automatically as soon as the vehicle starts moving, as the joystick is out of the neutral position. The safest and simplest way to stop your equipment is to release the joystick.

If faster braking is required, release the joystick completely or briefly move it in the opposite direction, then release once the wheelchair is stationary.

In the event of a power supply fault, the brakes are operated automatically.

If the control unit or joystick fails without a power failure, the wheelchair first stops (as if the joystick was released), then the brakes are operated automatically.

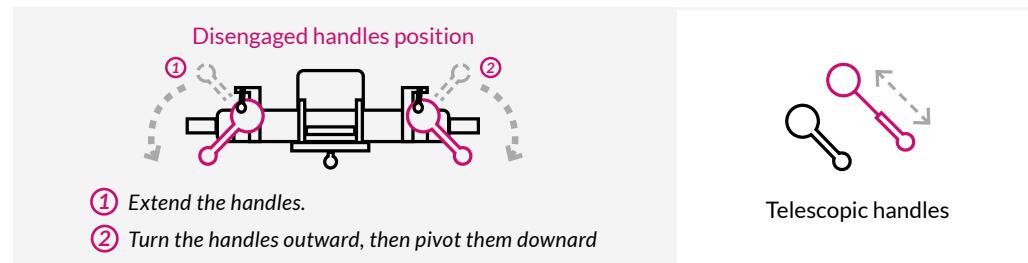
When you need to push the wheelchair for precise maneuvers or in case of emergency, the power add-on can be disengaged.

## HOW TO SWITCH TO MANUAL MODE ?

If needed, the wheelchair can be used manually without removing the power add-on.

Turn the handles to disengage the device.

Follow these steps:

# SERVICING AND MAINTENANCE

#### ⚠ CAUTION

It is important to follow the servicing and maintenance programme for the equipment to keep it in good condition.

#### ⚠ WARNING

If you're not sure you can service your equipment yourself, contact your trained dealer.

## SERVICING THE EQUIPMENT

- Before undertaking any cleaning, ensure the equipment is turned off.
- Clean the rollers and their housing regularly with a screwdriver.
- Once a week, run a vacuum cleaner around the motor, to remove all accumulated dust.
- Clean the entire power unit with a damp (not wet) cloth.
- Clean the electrical components and control unit with clean and dry cloth.
- Never use abrasive or alcohol-based detergents.
- Wipe the device immediately if wet or damp after cleaning or use.

#### ⚠ WARNING

Never wash your wheelchair using a hose or pressure washer

Only use common household cleaning and disinfection products, except abrasive or alcohol-based detergents.

#### ⚠ WARNING

Recommended tire pressure is 8 bar. It must never be below 6 bar.

Check tyre pressure regularly to ensure that it is always as close as possible to the recommended value. Use only the tyre brands and models recommended by BENOIT SYSTEMES and fitted at installation (SCHWALBE Marathon Plus or PRIMO Sentinel). Contact your dealer if you cannot find them.

Never replace the pneumatic tyres with solid tyres.

## PERFORMANCE CHECKS

After having cleaned or repaired the device, you must always check it is working correctly before using it:

- Turn on the controller – Do the indicator lights flash? If they do, this means there is a problem in the electronic system.
- Test the braking system on a flat surface with clear space of at least 1 m around the wheelchair: connect the control unit, check that the battery indicator remains stable. Push the joystick gently forward until you hear the parking brakes are disengaged (it is possible that the wheelchair will move forward), then immediately release the joystick. The brakes must then operate and stop the wheelchair.
- Repeat the test a further three times, pushing the joystick gently back, left and right.
- Activate all the electrical options, including the audible warning, to confirm they are working correctly.
- Use the device in all driving modes to confirm that it operates as before.

#### ⚠ CAUTION

Regularly check device condition to ensure safe use.

If in doubt about the performance of your device, contact your dealer.

It is recommended that the items shown in the table are inspected at the stated intervals.  
Your approved dealer should repair or replace any loose, worn, distorted or damaged parts before using your device.

Servicing and inspection programme	Daily*	Monthly	Quarterly	Annually
Check the battery charge indicator and the charger	●			
Check that the joystick on the control unit is neither distorted nor damaged	●			
Check that all the parts and devices are firmly attached	●			
Test the braking system	●			
Inspect the condition of the tyres and wheel fastenings	●			
Check that the cables and connectors are in good condition, are not tangled and are not in the way	●			
Inspect the condition of the seat you use		●		
Inspect all the bolts, nuts and fastenings to confirm they are not damaged or loose		●		
Check that the battery terminals are tightened			●	
Perform a full inspection, safety check and service by a BENOIT SYSTEMES approved dealer				●

\* Before each use

Correct servicing will improve your vehicle's performance and extend its service life, as well as reducing the risk of an accident.

## WASTE MANAGEMENT

Disposal and recycling must comply with applicable national and local regulations.

Protect the environment by recycling this product via appropriate channels:

- Power add-on and accessories (Waste Electrical and Electronic Equipment): to be disposed of at a waste collection center or to your dealer.
- Battery: to be disposed at waste collections centre.
- Packaging and packing materials: to be disposed of at a waste collection centre.

## TROUBLESHOOTING

The control unit dashboard is not lit or flashes	
Possible causes	Solutions
Battery incorrectly locked in place	Remove the battery slightly and re-engage it
Joystick wire poorly connected	Check the connection
Red connectors damaged or loose	Check that the connectors are in good condition and that the contacts are correctly positioned
Battery fault	Contact your dealer
Fault in a component of the power add-on	Count the number of LEDs flashing and contact your dealer
Power add-on was turned on when the joystick was not in the neutral position	Switch off and turn on again
A connection is faulty	Check terminals and connections
The dashboard lights up normally but the power add-on doesn't work	
Possible causes	Solutions
The power add-on is in the disengaged position	Engage the two handles
The power setting is too low	Increase the power
Rapid wear of the tyre treads	
Possible causes	Solutions
Poor quality tyres	Fit recommended tyres
Tyres under-inflated	Must not be below 6 bar
Insufficient pressure of the rollers on the tyres	Contact your dealer
Tyre worn on the inner wall	
Possible causes	Solutions
The roller cover is touching the tyre	Contact your dealer
The bracket for the wheelchair wheel is loose	Tighten this bracket correctly
The wheelchair goes in a circle	
Possible causes	Solutions
A manual brake of the wheelchair is on	Release it
A motor is faulty	Contact your dealer
The power add-on is making noise	
Possible causes	Solutions
Part of the wheelchair is touching a wheel	Visually check the condition of the wheelchair
A motor is faulty	Contact your dealer
A bearing is faulty	Contact your dealer
A roller cover is touching the tyre	Contact your dealer

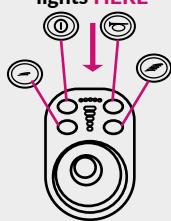
### A clutch handle is not staying engaged

Possible causes	Solutions
Incorrect adjustment of the ball joints	Contact your dealer
Distortion (following an impact for example)	Contact your dealer
<b>The power add-on does not unlock properly</b>	
Possible causes	Solutions
Clutch wire misadjusted	Contact your dealer

### CONTROL UNIT FAULT CODE INDICATOR

#### SMALL JOYSTICK FAULT CODE

To diagnose the fault, look at the **flashing lights HERE**

2 times	● ● ● ●	
3 times	● ● ● ●	Left motor faulty
4 times	● ● ● ●	Problem in left motor wiring
5 times	● ● ● ●	Right motor faulty
7 times	● ● ● ●	Problem in right motor wiring
8 times	● ● ● ●	Possible control unit problem
9 times	● ● ● ●	Possible problem in control system
		Possible brake problem

This diagnosis is only for guidance. Once the fault code is seen, please contact your BENOIT SYSTEMES trained dealer.

## TRANSPORT AND STORAGE

### TRANSPORT

#### By air

##### ⚠ ATTENTION

Before departure, contact the airline for all specific conditions and instructions to be followed. Check that removable parts are firmly secured or packed separately and labeled to avoid loss during loading/unloading. Take this manual with you.

#### By car

For safety reasons, the power add-on will be put in the boot of the car. The device must be removed when the manual wheelchair is used as a seat in a car. BENOIT SYSTEMES offers an optional charger for cigarette-lighter sockets.

##### ⚠ CAUTION

BENOIT SYSTEMES and its representatives refuses to accept any liability in case of accidents and/or damage on the wheelchair resulting from inappropriate use or failure to follow these instructions and/or applicable national regulations.

### STORAGE

If the device is not used for more than a week, you should prepare it for storage in order to extend its service life and the battery lifetime.

#### Follow these instructions:

- Store the device at a temperature of 15°C and avoid extreme temperatures in order to extend the service life of the vehicle and batteries.
- The components are tested and approved for the temperature ranges below:
  - \* The range of temperatures permitted for storing the device is between -40° and 65°C.
  - \* The range of temperatures permitted for storing the batteries is between -20° and 50°C.

##### ⚠ WARNING

Never store your equipment :

- Outdoors.
- In direct light.
- Close to a source of direct heat.
- In a damp environment.
- In a cold environment.
- With the batteries connected (even if the control unit is turned off)

##### ⚠ CAUTION

After an extended storage period at low temperature (-25°C), it is possible for the power add-on to turn itself off after being turned on; this situation may persist for about 2 minutes. In this case, place it at room temperature, wait for a few minutes, then try turning it on again.

For battery transport and storage conditions, refer to the battery user manual supplied.

# SAFETY AND HAZARD INSTRUCTIONS

## GENERAL SAFETY INSTRUCTIONS

### ⚠ WARNING

**Risk of damage to the equipment or serious, even fatal, injury if the control unit is used incorrectly**

- Undertake a test session with a healthcare professional before using a control unit.
- If you are not used to operate with a control unit, travel slowly and first familiarise yourself with the driving and directional characteristics of the wheelchair on a flat, unobstructed route.

### ⚠ WARNING

**Risk of damage to the equipment or serious, even fatal, injury.**

- Dropping lighted cigarettes onto a padded seating system can start a fire likely to cause damage to the equipment or serious, even fatal, injuries.
- The occupants of a motorised wheelchair are more particularly exposed to a risk of serious or even fatal injury associated with such fires and the resulting smoke, as they are not always able to get away from the wheelchair – Do not smoke when using this device.

### ⚠ WARNING

**Risk of damage to the equipment or serious, even fatal, injury.**

- Inappropriate supervision or servicing are likely to result in damage to the equipment or serious, even fatal, injuries, following breathlessness or ingestion of parts or materials.
- It is essential for children, pets or people with physical/learning disabilities to be supervised closely.
- Don't allow children or anyone else to use your device.

### ⚠ WARNING

**Risk of injury if the equipment is used for a purpose other than that described in this manual.**

- The device must be used in accordance with the manufacturer's instructions. It must not be used for a purpose other than that for which it is designed. Pay special attention to the safety instructions.

### ⚠ WARNING

**Risk of damage or injury if the vehicle is started unintentionally.**

- Do not sit on the motorised wheelchair during the adjustment phase.

- Disconnect the power supply from the device before getting onto it, getting off it or handling objects.
- Do not push the wheelchair by hand when the brakes are applied and when the power supply is disconnected.

### ⚠ WARNING

**Risk of falling from the motorised wheelchair.**

- Do not move forward on the seat, do not lean forward and do not stretch backwards beyond the top of the seat-back, such as to reach an object.

### ⚠ WARNING

**Risk of injury if driving under the influence of drugs or alcohol.**

- Never drive if you are under the influence of drugs or alcohol. If necessary, ask for help from someone else with appropriate physical and mental abilities.

### ⚠ WARNING

**Risk of injury if the contact is broken while the motorised wheelchair is moving, leading to the wheelchair stopping suddenly and violently.**

- Never operate the stop button nor disconnect a cable while the wheelchair is moving, except in an emergency.
- If you have to brake in an emergency, simply release the joystick to stop the wheelchair. Apply the wheelchair brakes on.

### ⚠ WARNING

**Risk of damage to the equipment or serious injury due to heat exposure.**

- Storing or using the vehicle close to a naked flame or flammable products can result in damage to the equipment or serious injuries.
- To ensure the equipment is reliable, avoid exposing it to extreme conditions, heat or humidity.
- Never store your device outdoors.

### ⚠ CAUTION

**Risk of injury if the maximum permitted load is exceeded.**

- Do not exceed the maximum permitted load (refer to the Technical Specifications section page 6).

- Check that the combined weight of the device and the user does not exceed the maximum permissible load of the manual wheelchair (refer to the technical specifications of the manual wheelchair).
- Never use the device to transport several people.

### ⚠ CAUTION

**Risk of injury in the event of inappropriate handling or fall of heavy parts.**

- During servicing and maintenance operations or when you are lifting certain parts of the equipment, consider the weight of each component, particularly the batteries. Ensure you always adopt the correct posture and don't hesitate to ask for help.

### ⚠ CAUTION

**Risk of injury caused by moving parts.**

- Ensure that the moving parts of the motorised vehicle, such as the wheels or joystick, do not cause injuries, particularly when children are present.

### ⚠ CAUTION

**Risk of fire or fault when connecting electrical equipment.**

- Do not connect your motorised wheelchair to any equipment that has not been explicitly permitted by BENOIT SYSTEMES. Rely on your trained dealer for all electrical installations.
- Do not use any other motorised device or accessory at the same time as power add-on. If you wish to use another device, please remove the power add-on from the wheelchair for your own safety.

## SAFETY INFORMATION ABOUT THE ELECTRICAL SYSTEM

### ⚠ WARNING

**Risk of damage to the equipment or serious, even fatal, injury.**

Inappropriate use of the motorised vehicle can result in the emission of smoke or sparks, and heat may be given off.

- The device must not be used for a purpose other than that for which it is designed.
- If the device begins to emit smoke, sparks or heat, stop using it and have it checked immediately.

### ⚠ WARNING

**Risk of injury or damage associated with shortcircuits.**

The pins of electrical connectors may remain live even when the system is switched off.

- Check that the cable insulation is sound and that the connectors are correctly plugged in, so that no live parts are accessible or in contact with conductive materials.
- Insulate the pins of the connectors when they are disconnected.

### ⚠ WARNING

**Risk of damage to the equipment or serious, even fatal injury due to moisture exposure or liquids, or corrosion of electrical components.**

- Reduce the exposure of electrical components to water and/or liquids.
- Electrical components damaged by corrosion must be replaced immediately.
- The power add-ons frequently exposed to water/liquids can require electrical components to be replaced more frequently.

### ⚠ WARNING

**Risk of serious, even fatal injury due to electric exposure.**

Failure to take note of these warnings can cause an electrical discharge resulting in serious or fatal injuries, or damage to the electrical system.

- The (+) battery terminal (red or brown cable depending on the model) must be connected to the red (+) power supply cable on the control box.
- The (-) battery terminal (black or blue cable depending on the model) must be connected to the black (-) power supply cable on the electronics box.
- Ensure to never connect the two poles of the battery (direct contact, tools or conductive material).
- Replace immediately cables with damaged insulation, frayed cables or cut cords.
- Do not disassemble or remove any battery connectors, covers or fuses.

## ⚠ CAUTION

### Risk of damage to the equipment due to electrical system failure.

A fault in the electrical system can cause unusual behaviour.

- In the event of a fault, turn the control unit off and back on again.
- If the problem persists, disconnect or remove the battery.
- If in doubt or for any question, don't hesitate to contact your dealer.

## SAFETY RECOMMENDATIONS ABOUT ELECTROMAGNETIC COMPATIBILITY

Electromagnetic fields, may have an influence on the operation of the vehicle. The electronic device used in this device can also cause weak electromagnetic disturbances, however these are below the legal limit. Therefore please consider the following comments:

## ⚠ WARNING

### Risk of incorrect operation following electromagnetic emissions

- Do not use a portable transmitter nor communication devices while the device is in use (radio devices or mobile telephones).
- Avoid going near powerful radio or television transmitters.
- If your wheelchair should start moving by itself, or if the brakes happen to be released, turn the device off.
- Report to your dealer all undesirable movements of the wheelchair that occur, even the electric brakes being released.
- Do not add any electrical or electronic accessories to the device.

## SAFETY INFORMATION ABOUT INSTALLATION

## ⚠ WARNING

### Risk of damage to the equipment or serious, even fatal, injury if improperly adjusted.

Prolonged use of the device with settings not complying with the specifications risk causing the vehicle to become unstable and causing damage to the equipment or serious, even fatal, injuries.

- Performance adjustments must only be made by people trained by BENOIT SYSTEMES who fully understand this process and the user's capabilities.

- After adjusting the device, ensure that it operates according to the parameters defined during the installation procedure.

If the device does not operate correctly, contact your dealer.

## ⚠ WARNING

### Risk of damage to the equipment or serious, even fatal, injury due to faulty mounting.

Inadequately tightened or missing fastenings can cause the vehicle to become unstable, resulting in damage to the equipment or serious, even fatal, injuries.

- After any adjustment, repair or servicing and before use, ensure that all the fastenings are fitted and suitably tightened.

## ⚠ CAUTION

### Risk of damage to the equipment or serious injury due to faulty control unit.

- Ensure that the control unit is securely mounted and that all screws are correctly tightened after adjustment.
- In the event of faulty control unit, immediately turn off the device's electronic system.

## ⚠ CAUTION

### Pinch points can arise when making adjustments. Be careful of your fingers.

## ⚠ WARNING

### Check all cables.

Damage caused to the cables of the wheelchair can cause safety problems and must be reported immediately to a trained technician.

## SAFETY INFORMATION ABOUT DRIVING

## ⚠ WARNING

### Risque of damage to the equipment or serious, even fatal, injury.

- Adapt your speed to the environment in which you are travelling. To control handling of the equipment, it is better to start on flat ground.
- Always ensure you can be seen from afar, especially if you use your motorised wheelchair at night.
- Pay close attention to traffic on the road.
- Don't forget that car or lorry drivers do not expect to see a wheelchair running at a speed greater than that achieved by propulsion. If there is the least danger, don't take the risk of crossing the road.

- The device can stop operating suddenly if a defect operates : we recommend using the seat belt when using the power add-on.

- Ensure to adjust the speed appropriately to safely overcome obstacles.

- Any malfunction of the control unit is liable to result in unexpected/erratic movements leading to damage to the equipment or serious, even fatal, injuries.

- If an unexpected/erratic movement occurs, stop using immediately the device and contact your dealer.

- Joystick or control unit must be replaced if it is damaged or weakened.

## ⚠ WARNING

### Risk of injury if the wheelchair tips over.

- Refer to the technical specifications (maximum safe slope).
- As far as possible, avoid driving on slippery, icy or oily surfaces when there is a risk of losing control of the vehicle. If you cannot avoid travelling over such surfaces, drive slowly and with the greatest care.
- Ensure that there are no objects in your path likely to damage the mechanism of your device.

- Do not cross an obstacle while moving forward.
- Avoid passing over thresholds or obstacles above the specified maximum height to not to threaten your stability. Remember to arrange for a slope, if necessary.

- Never try to go down a series of steps with your device.
- Avoid making sudden movements of the control unit od sudden changes of direction while the motorised vehicle is moving.

- Never use the motorised wheelchair to transport several people.
- Check that the user weight combined with the device do not exceed the maximum total permitted load (refer to the technical specifications of the wheelchair and the power add-on).

- Remember that the device brakes or accelerates when you change the speed of travel while the vehicle is moving.

- Never remove the anti-tip before or during the use of the device.
- **If your wheelchair is a standing model (e.g., LEVO Summit or equivalent):** never use the power add-on when the wheelchair is in the standing position. Always remove the control unit before moving into the upright position.

## ⚠ WARNING

### Risk of injury if you collide with an obstacle while passing through narrow passageways.

Pass through narrow passageways at the slowest speed of travel and with the greatest care.

## SAFETY INFORMATION

## ABOUT SERVICING AND MAINTENANCE

## ⚠ WARNING

### Risk of damage to the equipment or serious, even fatal, injury.

Incorrect repair or maintenance of this device by users/carers or by unqualified technicians are liable to result in damage to the equipment or serious, even fatal, injuries.

- Do not undertake any maintenance task other than those described in this servicing manual. These repairs and/or servicing must be carried out by a qualified technician. Contact a BENOIT SYSTEMES dealer or technician.

## ⚠ CAUTION

### Risk of accident if incorrect maintenance performed.

- For safety reasons and to avoid accidents related to passive wear, it is vital that this device should be checked once a year under normal use conditions.

## SAFETY INFORMATION ABOUT MODIFICATIONS MADE TO THE EQUIPMENT

## ⚠ CAUTION

### Using unapproved components or accessories is liable to result in injuries or to damage the motorised vehicle. Using accessories and parts not approved by BENOIT SYSTEMES may alter the stability of the motorised wheelchair and increase the risk of tipping over.

- Use only accessories and parts approved by BENOIT SYSTEMES for this equipment.
- Electrical and electronic components not approved by BENOIT SYSTEMES for this device are liable to cause fires and electromagnetic damage.
- Use only electrical and electronic components approved by BENOIT SYSTEMES for this equipment.
- Ensure you always quote the serial number of the equipment when you order spare parts.

## IMPORTANT INFORMATION ABOUT THE MAINTENANCE TOOLS

Certain maintenance tasks described in the servicing manual can be performed by the user without problems require the use of appropriate tools. If you don't have these tools, we advise you not to perform these maintenance tasks. If necessary, we recommend contacting your dealer.

## CE MARKING

Conformity was assessed and CE marking applied in compliance with European Regulation 2017/745/EEC and only applies to the complete product. CE marking is invalidated if components or accessories are replaced or added without having been approved by BENOIT SYSTEMES for this product.

## IMPORTANT LEGAL NOTICE FOR THE USER OF THIS PRODUCT

Incidents due to device malfunctions and resulting in serious physical injuries must be reported to the producer and to the competent authority of the Member State where the user is located.

### IN FRANCE

- Manufacturer: BENOIT SYSTEMES, 7 rue du Pont 21450 BILLY-LÈS-CHANCEAUX
- Competent authority: ANSM, 143 Boulevard Anatole France 93200 SAINT DENIS

### IN THE MEMBER STATES OF THE EUROPEAN UNION

- Your distributor
- National authorities responsible for incidents with medical devices.

You will find an overview of the responsible authorities by following the link: <http://ec.europa.eu/growth/sectors/medical-devices/contacts/>

## GUARANTEE

The guarantee period for the power add-on, battery and charger is 24 months from the invoice date.

All spare parts come with a 6-month warranty from the date of delivery, while all batteries purchased separately are covered by a one-year warranty.

To claim against the guarantee, just make a note of the serial number of your equipment shown on the first page of this manual and contact your approved dealer or to register on our website. The serial number is also under the power add-on, between the battery brackets.

## CONDITIONS

The guarantee is only valid if the product is used under the specified conditions and for the use for which it is designed.

The guarantee may not be applied if the product requires repair or replacement for one of the following reasons:

- The power add-on, or one of its parts, has not been maintained according to the manufacturer's recommendations, as stated in the User Manual, or parts other than those specified for this equipment are used.
- The power add-on, or one of its parts, was damaged as a result of negligence, an accident or inappropriate use
- The power add-on, or one of its parts, has been converted or a repair made before having notified the after-sales service department.

This guarantee is limited to the free-of-charge repair or replacement of parts recognised as faulty in the manufacturer's workshops or those of its representative (including labour and carriage). This repair may be performed using new parts or standard exchange parts, depending on the assessment made by the manufacturer or its representative.

Work carried out under the guarantee does not extend the guarantee.

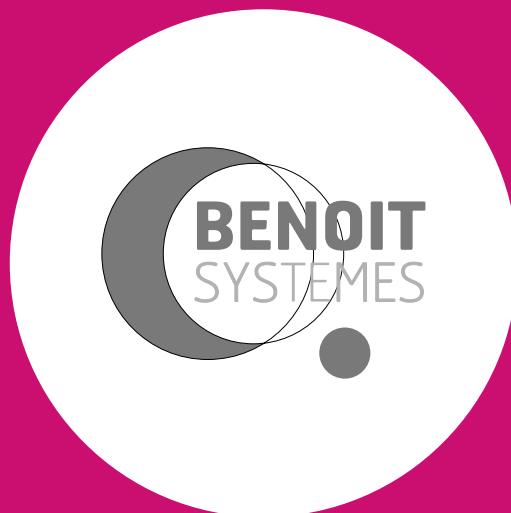
## LIMITS OF LIABILITY

BENOIT SYSTEMES declines all liability for damage related to:

- Failure to comply with the user manual.
- Incorrect or improper use of the equipment.
- Normal wear and tear.
- Inappropriate storage of the equipment.
- Incorrect assembly or dismantling by the user or other parties.
- Technical modifications on and in the equipment.
- Unauthorised modifications and/or use of spare parts not manufactured or recommended by BENOIT SYSTEMES.

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This medical device is a regulated health product and bears the CE marking. As manufacturer, BENOIT SYSTEMES declares that this product is Class I medical devices and complies with EU Regulation 2017/745/CEE on medical devices.